



AlcoDigital Platinum

USER GUIDE

The AlcoDigital Platinum uses fuel cell technology with highly-accurate results and user-friendly design.

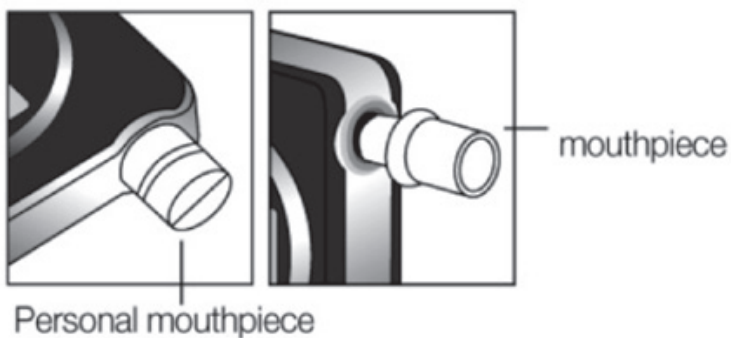

Package contents: AlcoDigital Platinum breathalyser
AAA batteries • Hard case • Soft zip pouch • Mouthpieces • User guide

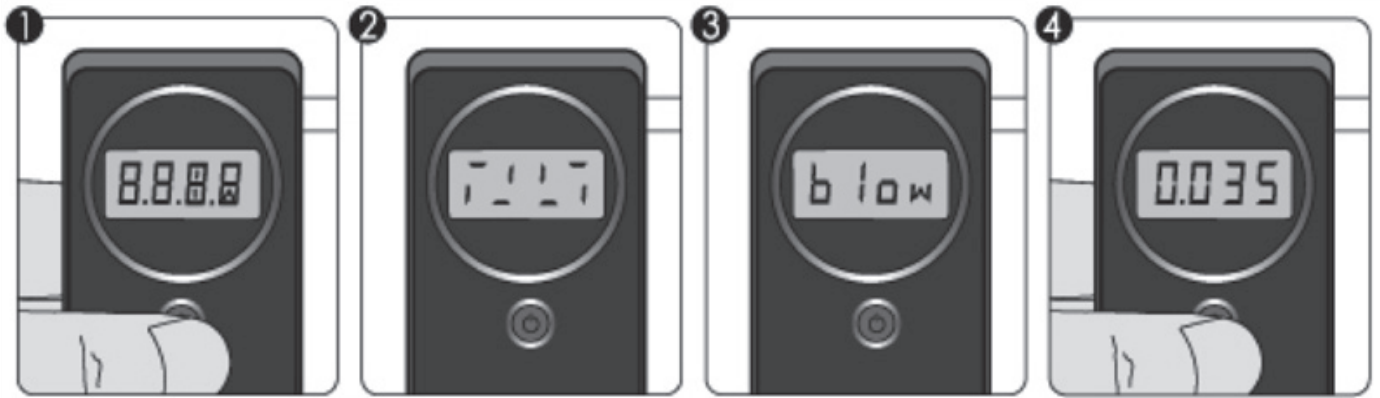

1. Press and hold power button for 2 seconds to switch on.
2. Wait for sensor warm-up till 'blow' is displayed.
3. Take a deep breath and blow slowly and consistently through the mouthpiece for approx 4 seconds until tone stops.
4. Result will be displayed for 15 seconds; to re-test press power button.
5. Switches off automatically after 4 minutes.
6. To check test count, press and hold power button for 4 seconds.

SPECIFICATION

Indication of B.A.C. (Full 3 decimals)	0.000 to 4.000 ‰ or 0.000 to 2.000 mg/l or 0.000 to 0.400 %BAC by LCD Display
Warm up time	5 seconds - 4 minutes
Response time	Within 10 seconds at 0.10 %BAC
Sensor	Fuel cell sensor
Power supply	Two 1.5v AAA size alkaline batteries
Weight	85g (Including Batteries)
Dimensions	105mm (H) x 50mm (W) x 15mm (D)
Ambient conditions	Operation : 5 to 40°C Storage : 0 to 50°C
Accuracy	±0.005 %BAC at 0.050 %BAC
Sensor module	Sensor module should be replaced after approximately 500 tests or if damaged by improper testing or handling.

WARNING DISPLAY

Flow: The breath sample was not strong enough for measurement. After warming automatically, please blow strong and steady during sampling.

bAt: The battery power is low. Please replace the batteries.

CAL: The cartridge should be changed at least every 12 months or every 500 tests. If the display shows "CAL" for 3 seconds prior to warm-up, the results may not be reliable the cartridge should be replaced as soon as possible.

Err6: Sensor module empty – Please check sensor module.

Err9: System error – Please repair the device.

Err2: Sensor module error – Please replace sensor module.

PERSONAL MOUTHPIECE

Push the hook aside like below to use personal mouthpiece.



**Call 0208 454 7372 for expert advice from AlcoDigital,
the UK's Top Breathalyser Specialists**

Email: sales@AlcoDigital.co.uk www.AlcoDigital.co.uk

AlcoDigital makes no warranties, expressed or implied, as to the ability of this device to determine whether a person is intoxicated, and expressly disclaim any liability for incidental, special, or consequential damages of any nature. Decisions and/or actions based upon the reading of this device shall be at such person's own risk. AlcoDigital assume no responsibility for consequences of subjects who test negative when using this device and later show that they are under the influence of alcohol or their judgment has been impaired by alcohol. This device should only be used as a screening device and may only give an indication of the possible presence of alcohol in the blood of the test subject. Correlation between breath alcohol content and blood alcohol concentration depends on many variables, including temperature and health conditions. A safe or low reading on a breath alcohol screener does not mean that the driver's reaction times can respond to any emergency encountered. The exact concentration of alcohol in the blood of the test subject cannot be exactly determined by using a breath alcohol screening device.

HOW TO REPLACE SENSOR MODULE

1. Open battery cover and remove batteries.
2. Squeeze catch on left to release sensor module and slide DOWN to disengage.
3. Lift out sensor module and insert replacement.
4. Slide replacement sensor module UP into place and ensure catch locks securely.

NB: DO NOT ATTEMPT TO REMOVE SENSOR MODULE WITHOUT ENSURING CATCH IS RELEASED AND MODULE IN DOWN POSITION.

